


# ESPELUX Edu Installation Guide [Mac]

ESPELUX Edu Download Preparation Materials

Please insert the DVD-ROM and Type-A USB to the PC.

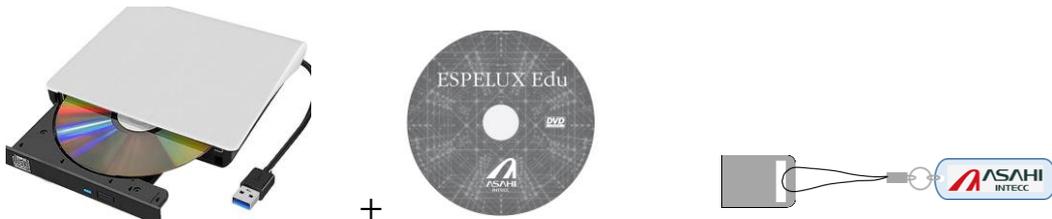

## Step1: Installing CodeMeter Runtime


Please install CodeMeter Runtime (license authentication tool, first time only)

\*The app cannot be started on a PC on which it is not installed.

If you change your PC, please be sure to install it for the first time.


① Start the installer by double-clicking

[CodeMeter-Runtime-Installer/CodeMeterRuntime-7.60d-mac.dmg]

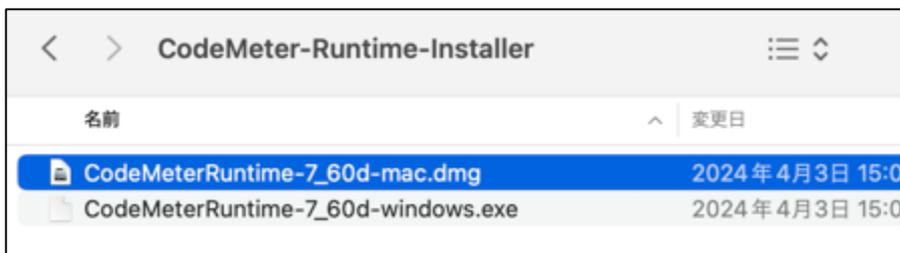


② Installing

1. Double-click "CmInstall.pkg"

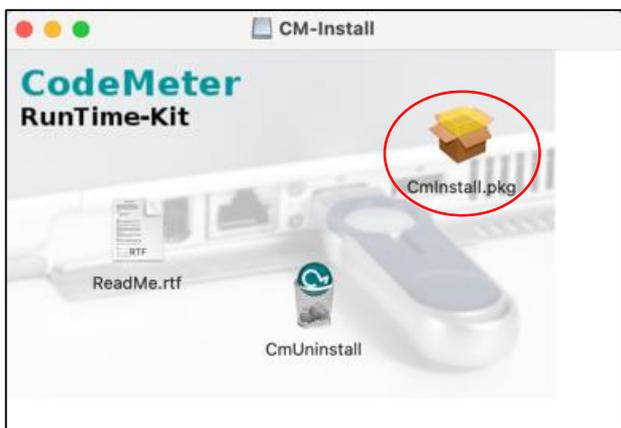

## 2. Click "Continue"

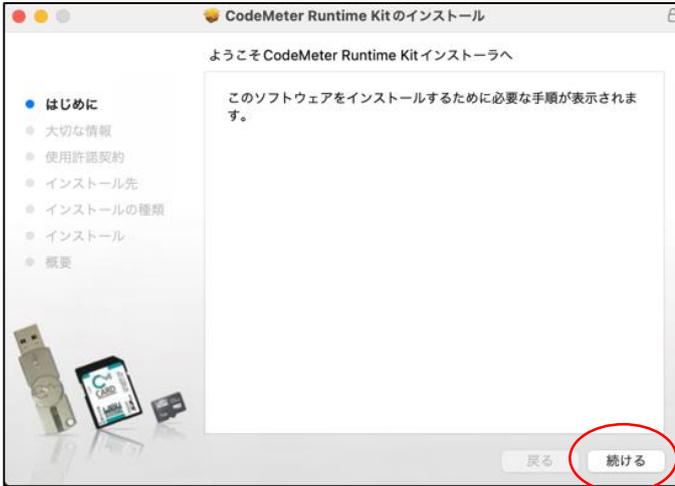

## 3. Click "Continue"



## 4. Read the license agreement and click "Continue"



## 5. Click "Agree"



## 6. Click "Continue"



## 7. Click "Install"



8. Enter the password and click "Software"

Click "Install"




9. When the installation is complete, click "Close"



## Step2: Installing ESPELUX Edu


① Double-click the installer

[Mac/ESPELUX-Edu-1\_0\_0.dmg]

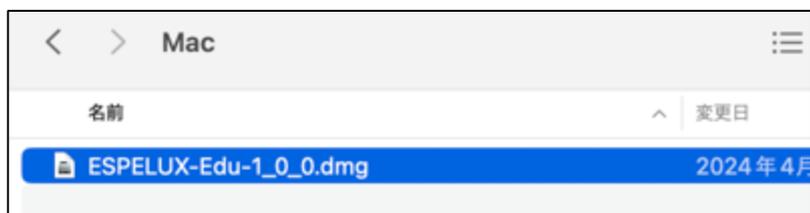


## ② Put ESPELUX Edu in the Applications folder

Drag and drop

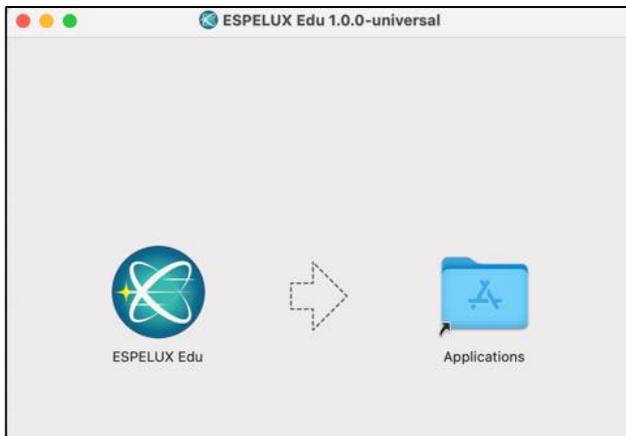


By dragging and dropping, the app is installed and stored in the Applications folder

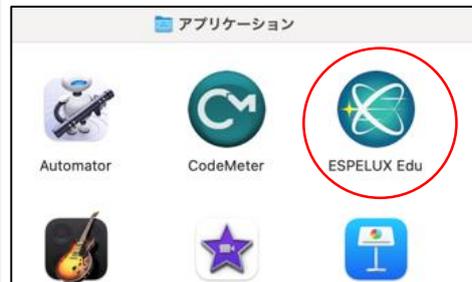

## Step3: Start ESPELUX Edu

You can start ESPELUX Edu by double-clicking it in the Applications folder.

Or search for "ESPELUX Edu" in **Spotlight search**. You can launch ESPELUX Edu by double-clicking.

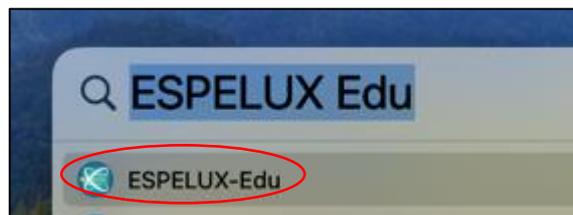

## Important Notes (Frequently Asked Questions)

- Display resolution:

**MacOS: 1680×1050 or higher, recommended to be within 1710×1112**


- When measuring:

**Please measure both View1 and View2 as Ve1.**

# ESPELUX Edu Installation Guide [Windows]

ESPELUX Edu Download Preparation Materials

Please insert the DVD-ROM and Type-A USB to the PC.

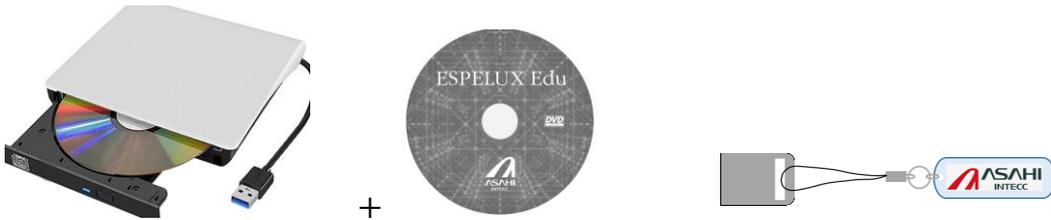

## Step1: Installing CodeMeter Runtime


Please install CodeMeter Runtime (license authentication tool, first time only)

\*The app cannot be started on a PC on which it is not installed.

If you change your PC, please be sure to install it for the first time.

### ① Start the installer by double-clicking

[CodeMeter-Runtime-Installer/CodeMeterRuntime-7.60d-windows.exe]

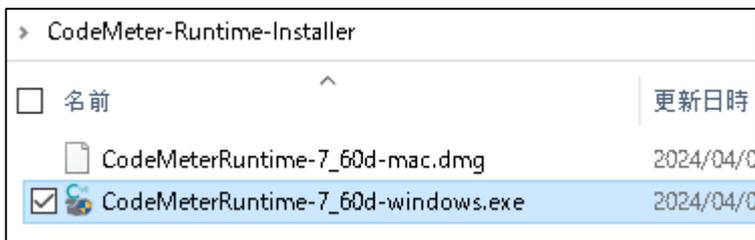


### ② Installing

#### 1. Click "Next"




## 2. Please read the license agreement


Check "I agree" and click "Next"

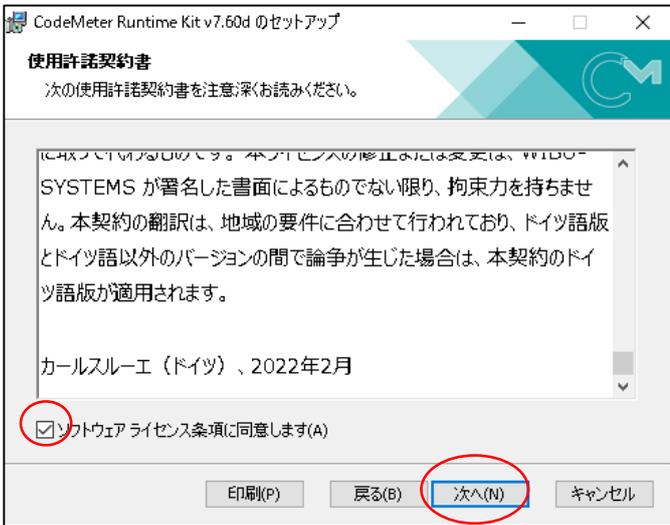

## 3. Click "Next"

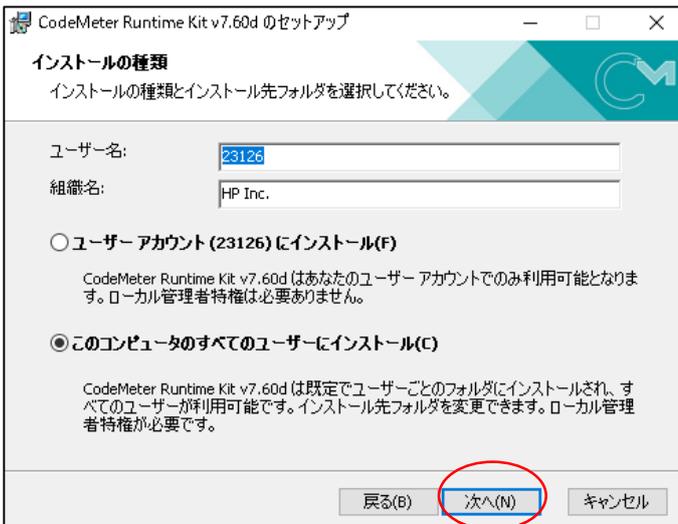

## 4. Click "Next"

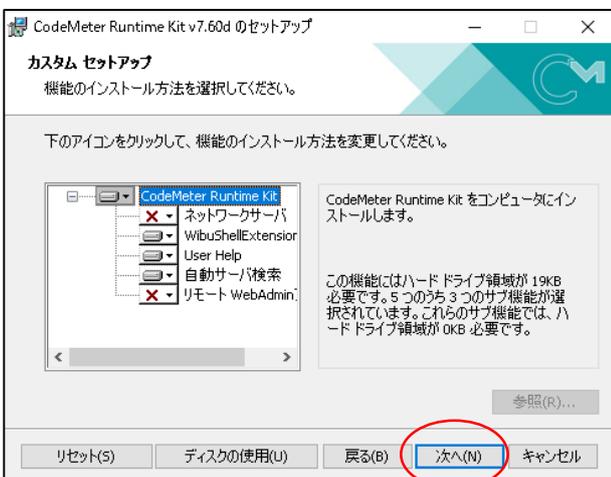

## 5. Click "Install"





## 6. Click "Finish"



## Step2: Install ESPELUX Edu

① Install by double-clicking the installer

[Windows/ESPELUX-Edu-1\_0\_0.exe]

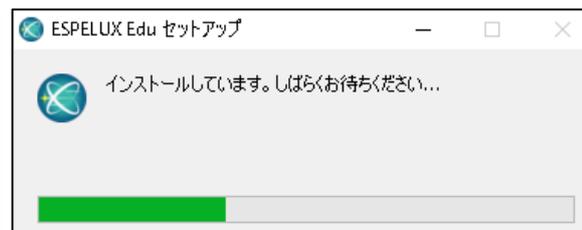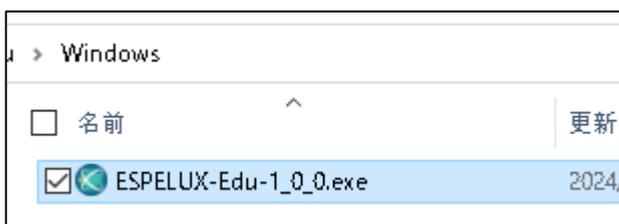



The app will start when the installation is complete.

### Step3: Start ESPELUX Edu

A shortcut will be created on your desktop during the initial installation.

From the second time onward, you can start the shortcut by double-clicking it.



Alternatively, you can search for "ESPELUX Edu" from the start menu and launch it.



## **Important Notes (Frequently Asked Questions)**

- Display resolution:

**Windows: 1920×1080 (100%) recommended**

- When measuring:

**Please measure both View1 and View2 as Ve1.**